


QUICK REFERENCE GUIDE

AZO®

Test Strips

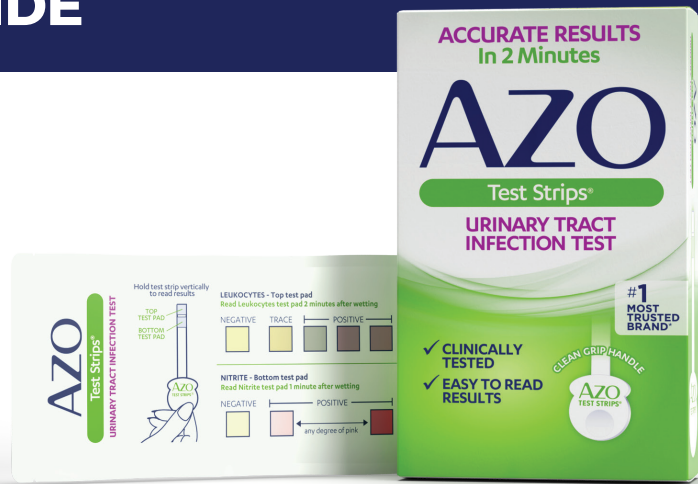

Product Name & Description	AZO® Test Strips Urinary Tract Infection Test
Intended Usage	At home urine test for diagnosing a UTI
Test Sensitivity	Leukocyte and nitrite test pads
Principle	Each test strip has a pad for both Leukocytes (white blood cells) and Nitrite. Clean grip handle for ease of use and color chart to interpret results
Directions for Use	Simply wet one test strip by holding it in the urine stream for 1-2 seconds. Read result at 1 minute for Nitrite and at 2 minutes for Leukocytes. Refer to insert for full directions.
Where to Buy	

LEUKOCYTES - Top test pad
Read Leukocytes test pad 2 minutes after wetting

NEGATIVE **TRACE** **POSITIVE**

NITRITE - Bottom test pad
Read Nitrite test pad 1 minute after wetting

NEGATIVE **POSITIVE**

any degree of pink

Hold test strip vertically to read results

AZO® and AZO Test Strips® are trademarks of DSM.